

Mobile Medical Oxygen Delivery

With innovative cylinder technology



Enhanced cylinder features
with proven reliability

- Increase patient safety
- Optimize time
- Improve efficiency

Always there to simplify gas management

INTELLI-OX™ is an innovative medical gas solution that combines Air Liquide's patented digital gauge with a lightweight aluminum cylinder, integrated valve and regulator, and ergonomic handle to comprise one of the industry's safest and most reliable medical oxygen USP delivery system.

INTELLI-OX also gives caregivers accurate time remaining for medical oxygen in the cylinder without having to interpret pressure data—simplifying gas management and reducing gas waste—so caregivers can focus on patient care instead.

Caregivers trust that INTELLI-OX™ is certified MR Conditional to 3.0 Tesla, when located outside the MRI bore

Audible alerts inform caregivers when the cylinder is almost empty to help prevent run out*

Comprehensive medical oxygen information helps caregivers ensure compliance for cylinder storage and patient care

An innovative mobile cylinder with enhanced features and proven reliability

Use accurate information to improve patient care



Increase patient safety

- Clear display of the remaining time and volume available indicates precisely when the cylinder needs to be replaced.
- Visual and audible alerts are triggered when there is ¼ content remaining and 15 minutes remaining. When there is ¼ content remaining at the current flow rate, an audible 3 beeps sound alert is triggered. Time remaining will still be displayed. When there is less than 15 minutes remaining at the current flow rate, an audible 2 beeps sound alert is triggered and 4 dashes (---) are displayed. Pressure bar graph, hourglass and dashes flash continuously.*
- Automatic calculation of time remaining allows caregivers to focus on their core objective of ensuring patient safety.

Optimize time


INTELLI-OX™ provides fast and easy medical oxygen set-up for patients

- Integrated valve and regulator assembly means no more time is wasted hooking up a regulator.
- Caregivers can quickly select a cylinder that meets their requirements the first time.

Improve efficiency

INTELLI-OX gives caregivers clear information to better manage medical oxygen

- Caregivers can safely use more medical oxygen from each cylinder.
- With proper utilization, facilities can reduce the frequency of cylinder turnover and associated costs including time spent on cylinder management, handling, transportation, and order management.

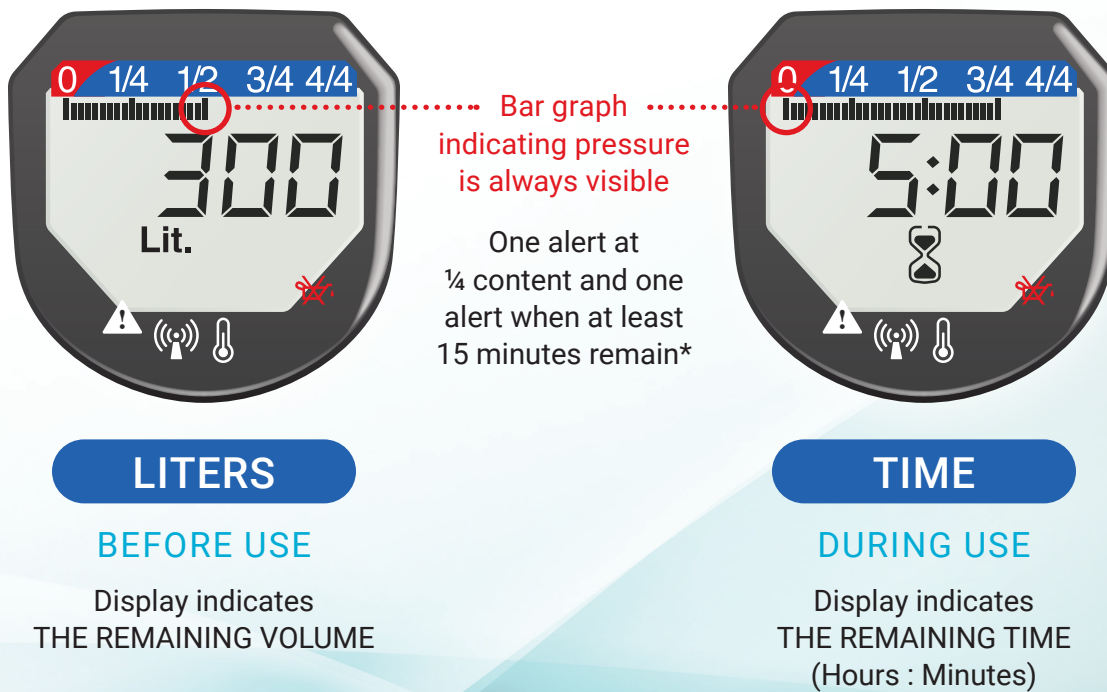

*Please note that ambient and background noise may affect the alert's audibility. Hearing impairments may also affect an individual's ability to detect the alert. The visual display on the digital gauge (rather than the sound alert which is set at particular intervals) provides the primary and real time method to determine the amount of medical oxygen remaining by displaying: (1) the pressure bar graph (2) the volume of gas in liters and (3) the time remaining calculation when the cylinder is in use.

Air Liquide Healthcare

Changing care. With you.

Always There

Contact

www.airliquidehealthcare.ca

Customer Service: 1-833-935-3252

Ontario: medgas.on@airliquide.com

Quebec: medgas.qc@airliquide.com

Atlantic: medgas.atl@airliquide.com

Western Canada: medgas.west@airliquide.com

MCM-459-CA | ©2025 Air Liquide Canada Inc.

